



CS-Pro VET

Equine Protocol Recommendation

Protocols

1. External Occipital Protuberance

Standoff: 20 mm
EFD: 0.19 mJ/mm²

2. Cervical Articular Facets

Standoff: 30 mm
EFD: 0.22 mJ/mm²

3. Back Pain

Standoff: 30 mm
EFD: 0.22 mJ/mm²

4. Medial Meniscus of the Stifle

Standoff: 30 mm
EFD: 0.22 mJ/mm²

5. Gluteal Trigger Points

Standoff: 30 mm
EFD: 0.22 mJ/mm²

6. Scapulohumeral Joint

Standoff: 30 mm
EFD: 0.22 mJ/mm²

7. Bucked Shins

Standoff: 10 mm
EFD: 0.18 mJ/mm²

8. Proximal Suspensory Ligament

Standoff: 30 mm
EFD: 0.22 mJ/mm²


9. Lower Hock Joint

Standoff: 30 mm
EFD: 0.22 mJ/mm²

10. Foot Pain

Standoff: 20 mm
EFD: 0.19 mJ/mm²

For more information, contact
sales@curative-sound.com

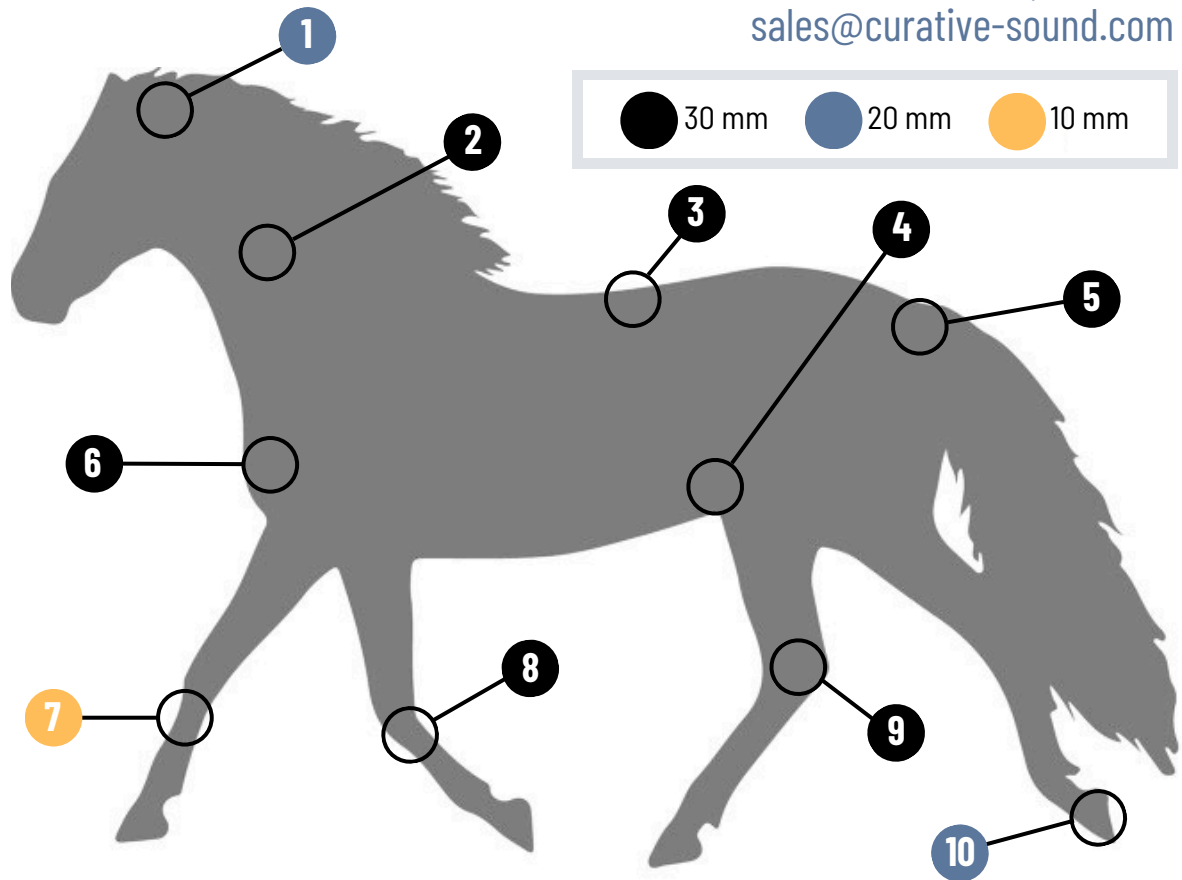

General Recommendations

Treat each area with 3000 pulses at a repetition frequency of 10 Hz. Conduct treatment every 3-10 days for a total of 3-6 treatments.

These recommendations are only general guidelines and can be deviated from at veterinary discretion.

Other Treatments

Tendon and Ligament Injuries

Standoff: 10 mm
EFD: 0.18 mJ/mm²

Splints

Standoff: 10 mm
EFD: 0.18 mJ/mm²